

Instruction for Use

Clean 'n Easy Interior

Intended Use:


Cleaning of non-invasive medical devices and other hard surfaces in healthcare facilities, nursing homes, retirement homes etcetera.

How to use:

1. wash your hands,
2. put gloves on (for hygienic reasons) *
3. open the dispensing lid on the bucket and pull out a cleaning towel,
4. close the dispensing lid,
5. clean the surface with the ready to use towel, if the towel gets to dry, take a new towel out of the dispensing lid,
6. throw away the used towels in the waste bin
7. take off gloves and throw them away in the waste bin
8. wash your hands

* change gloves between rooms to prevent cross-contamination

This product is meant for single-use.

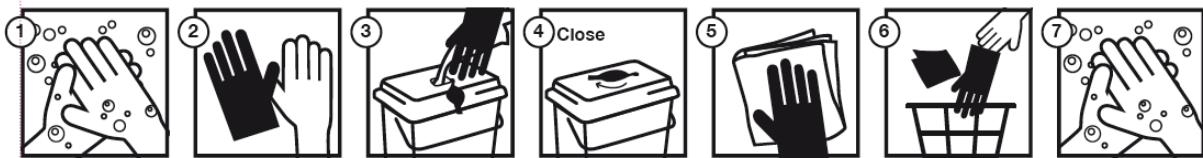

Safety:

The product is not classed as dangerous.

Always read the MSDS

After inhalation : if excessive inhaled allow the person to rest in a place with fresh air

After skin contact : rinse off with water

After eye contact : remove contact lenses if present. Rinse thoroughly with water.

Consult a physician if unwanted effect or irritation develops

After swallowing : not applicable

Variations

02036000	Clean 'n Easy Interior	(1x150 towels)
02036010	Clean 'n Easy Interior refill	(3x150 towels)



Storage temperature	: 5-30°C
Shelf life	: see label on roll